

Carpus Paw Fitting



Fitting Video:


<https://vimeo.com/album/2461730/video/71217897>




1. Holding the brace on its side, place the paw through the hole




2. Rotate the brace up. Place the middle toes on the edge of the paw piece.

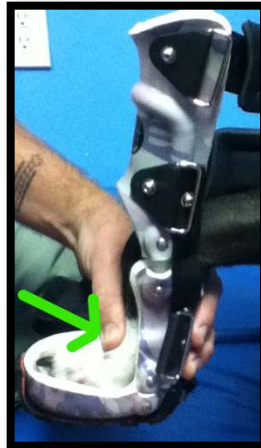

3. Pushing down and back on top of the paw, place the main paw pad in the back corner of the segment.

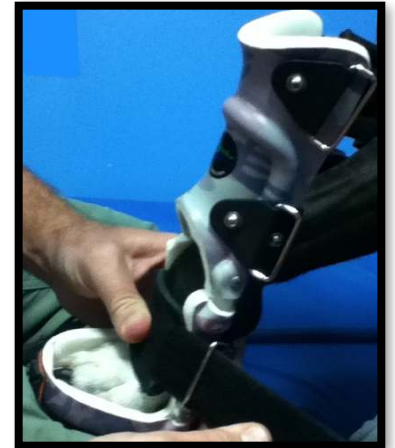

4. While holding the paw in the desired position, fasten the bottom strap tight. This is a very important strap and needs to be snug.

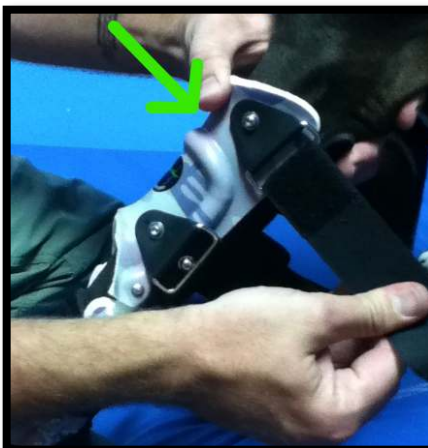

5. Push the top part of the device onto the leg. While holding in this position, fasten the top strap snug.



6. Fasten the middle strap while holding the device against the limb. Ensure your bottom 2 straps have the pads with the rolled edge down, and the top pad has the rolled edge placed upwards.



7. The accessory carpal pad should be resting above the foam.



8. The paw should be seated down in the paw segment. The toenails may be touching the very front or a small amount of space may be seen.