

CARPUS PAW FITTING INFORMATION



FITTING VIDEO: [HTTPS://VIMEO.COM/70487062](https://vimeo.com/70487062)



STEP 1: Hold the device with one hand and with the other hand place the middle two toes on the edge of the paw segment.



STEP 2: Push **back and down** on the paw to slide into the paw segment. The nails should be positioned at the front of the paw segment and may touch the foam. **THIS STEP IS VERY IMPORTANT FOR PROPER ALIGNMENT OF THE ENTIRE DEVICE.**



STEP 3: With one hand hold the bottom strap and paw in place within the device. With the other hand, thread the strap through the buckle and firmly secure Velcro in place. Be certain the strap pad is centered on the leg.



STEP 4: With one hand hold the top strap and leg in place within the device. Thread the strap through the buckle and firmly secure Velcro in place. Be certain the strap pad is centered on the leg.



STEP 5: Secure the remaining middle strap(s) in the same manner.



CORRECT

paw alignment in the device