

# Tarsus Paw Fitting

Please watch this video on how to fit your Tarsus Paw orthotic:

<https://vimeo.com/album/1924356/video/41465106>



1. Extend hip and knee
2. Place middle two toes on lip of paw segment
3. Push paw into segment
4. Make sure main paw pad is resting in paw segment



5. Hold leg, strap, and brace together like a C-clamp with your hand.
6. While in that position, fasten the bottom strap snug



7. Align vertical tabs of device parallel with anatomical structure of leg, the tibia and fibula



8. Gently squeeze the brace inward to touch the leg, and then fasten the top strap.



9. Fasten back top strap



10. Fasten two front straps

***Please note that you will see space in-between the calcaneus and brace, this is to prevent irritation.***