

Tarsus Fitting

Fitting video:

<https://vimeo.com/70802982>



Step 1: Place the foot through the opening at the hinges. The brace will need to be perpendicular to the limb.



Step 2: After rotating the brace onto the limb, align the bottom of the device just above the main paw pad, and visualize the calcaneus.



Step 3: Hold the device against the leg and fasten the bottom strap very snug.



Step 4: Fasten the next strap up snug.



Step 5: Push the top of the device onto the leg and fasten the top strap snug.



Step 6: Gently pull on the neoprene strap and fasten around the Achilles tendon.

Please email ground level pictures of your pet in the device from the back, side, and front to ifo@walkingpawsrehab.com